

## ORDER OF DRAW CHART

**TUBES MUST BE DRAWN IN ORDER OF SEQUENCE 1 THROUGH 10 (RED → BLACK) AS REQUIRED**

Red Section I	Light Blue Section II		Yellow Section III				Green (Na Hep.) Section IV	Green (Li Hep.) Section V	Lavender Section VI	Gray Section VII	Pink Section IX	Black Section X
1 tube each	1 tube for all:	1 tube each (max 2):	1 tube		Extra tube		1 tube each	1 tube each	1 tube each	1 tube each	1 tube each	X
Cold Agglutinins	PT/INR	AT3	AFP	Estradiol/E2	Phenobarbital	ANF	Chromosome Studies	Amino Acid	CBC/ DIFF/ RETIC	Glucose Tolerance Test/GTT	Blood Group Coombs	ESR
C-Peptide	PTT	Factor Assays	Albumin/ALB	Ferritin	Phosphate	Cardiolipin Abs		Organic Acids	Hemoglobin Electrophoresis <i>10 ml minimum</i>			
Cryoglobulin <i>maintain ambient temperature</i>	KCT	Heparin	ALP	Folate	PreALB	Ionized CA/iCa	Osmotic Fragility <i>Book ahead</i>	Aluminum		Lactate <i>on ice</i>		
Sertraline (Zoloft)	D-Dimer	Protein.C	ALT	FSH	Progesterone	Protein		Chromium	Sickle Cell Screen			
	Fibrinogen	Protein.S	Amylase	Gentamicin	Prolactin/PRL	Electrophoresis		Lead	Kliehauer-Betke			
	HIT assay <i>send yellow tube also</i>		AST	GGT	Protein, Total	PINV/SPE/IFE		Mercury	Malaria smear			
			Bilirubin	Glucose	PSA, fPSA	Mono Test		Zinc	G6PD			
			Total/Conjugated /B2M	HCG	Parathyroid Hormone/PTH	ASOT			Lymphocyte Sub Sets <i>Immunology Requisition</i>			
			Beta 2 Microglobin	Haptoglobin	Rheumatoid Factor/RF	Maternal Serum Screening/MSS			Factor V Leiden/ Prothrombin variant <i>DNA requisition</i>			
			IgA	IgG	Salicylate	Osmolality			Carboxyhemoglobin			
			Calcium	IgM	Theophylline	Viscosity			Methemoglobin <i>on ice</i>			
			Carbamazepine	IgE	Testosterone	Acetaminophen			Ammonia <i>on ice</i>			
			CEA	Iron	SHBG	Amitriptyline			Cyclosporin/CYA			
			Cholesterol	TIBC	cBAT	Clomipramine			Tacrolimus/FK506			
			CK	% Saturation	Thyroid Abs	Clonazepam			HbA1C			
			CO2	Lipids:	Tobramycin	Desipramine			RBC Folate			
			Cortisol	Chol/HDL	Troponin	Imipramine			Homocysteine <i>on ice</i>			
			Creatinine	LDL/Trigs	Transferrin	Lithium/Li			Renin (pre-chilled)			
			eGFR	LH	Triglycerides	Methotrexate						
			CRP/hsCRP	Magnesium	TSH	Nortriptyline						
			Digoxin		FT4	Trimipramine						
			Dilantin		Urea/BUN	Vancomycin						
			Electrolytes /Na/K/Cl		Uric Acid							
					Valproic Acid							
Tubes must be spun within 4 hr	Tubes must be spun within 4 hr		Tubes must be spun within 4 hours				Osmotic Fragility must be prebooked		Homocysteine must be spun within 1 hr			

**Section III:** Any combination of testing can be done on 1 tube, **EXCEPT** for the tests in the yellow column (2 tubes).